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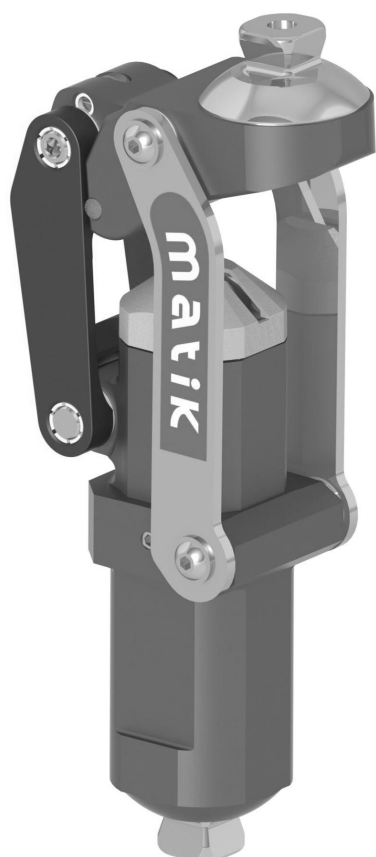
ASSEMBLING AND ADJUSTMENT INSTRUCTIONS FOR 1P200/1P200-KD PNEUMATIC KNEE

Contents :

One 1P200 or 1P200-KD pneumatic knee

Use :

This knee is designed exclusively for lower limb prosthesis.

 It is specifically recommended for active patients and low activity level patients, up to 125 kg (carried load included).

Features :

In extension this joint system with short linkages puts the instantaneous center of rotation backward the load line. This ensures a good stability in stance phase. Adjustments (E) and (F) allow to adapt shock absorption during flexion independently from shock absorption at the end of extension. The extension assist is not adjustable.

Weight : about 743 g.

Possible flexion up to 170°. Flexion is only limited by the contact of the knee with the socket or the cosmetic cover.

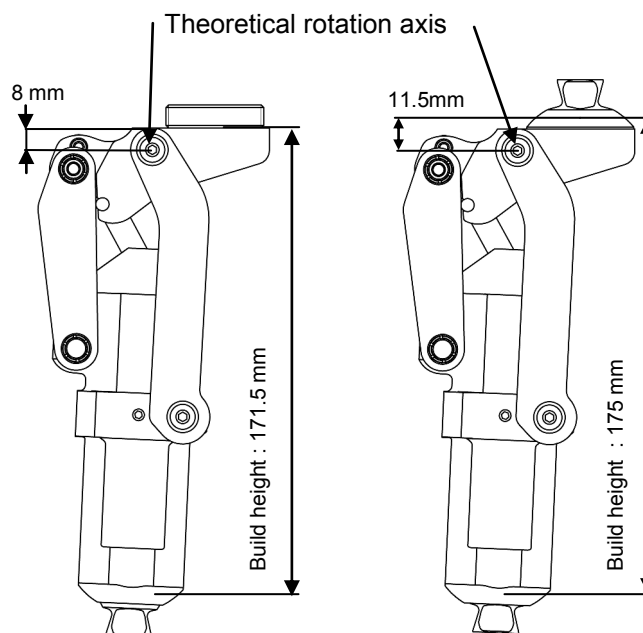
Assembling : Connections with the other prosthetic components.

This knee is available in two different models.

1P200: *Upper connection* by a male pyramid that is convenient for TF amputation or hip disarticulation. *Lower connection* by a male pyramid connector.

1P200-KD: *Upper connection* by a connector with a M36x1,50 screw that is convenient for long stump or knee disarticulation. *Lower connection* by a male pyramid connector.

FINISHING: 1G13 one-part cosmetic cover (complete or partial).

1P200-KD
195 mm

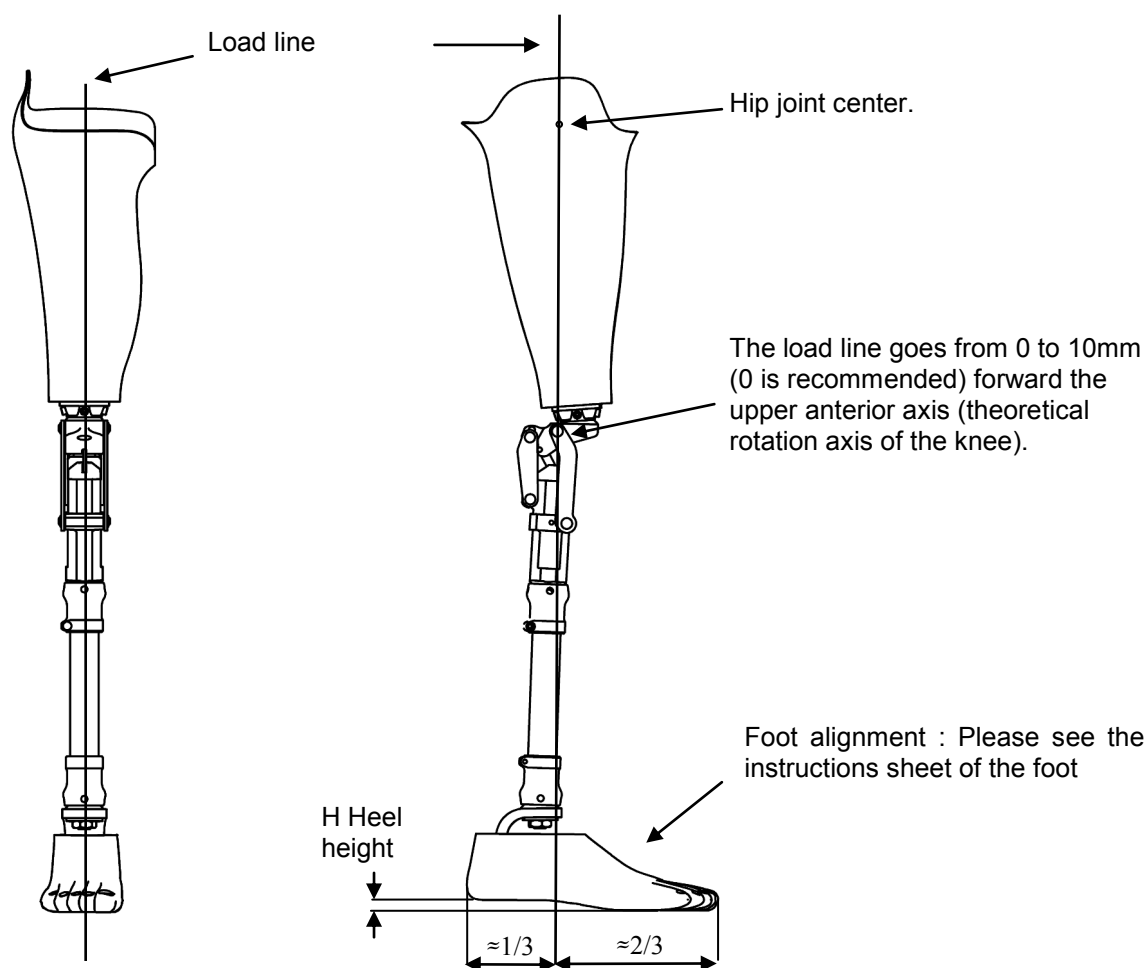
Total height:

1P200
204 mm

 **Inform your patient:** The warranty is not applicable to damages due to a bad assembly, an inadequate alignment, a very dusty environment without adapted protection, or any inappropriate use.


 Using a cosmetic cover is highly recommended to prevent your fingers or clothing from being jammed in the joint. Avoid exposing the knee to situations that may cause the corrosion of the metallic parts (fresh water, sea water, acid, ...). In case of fall, shock or abnormal behaviour of the joint, contact your CPO for knee control.

Alignments recommended when assembling the prosthesis

Stability adjustment.

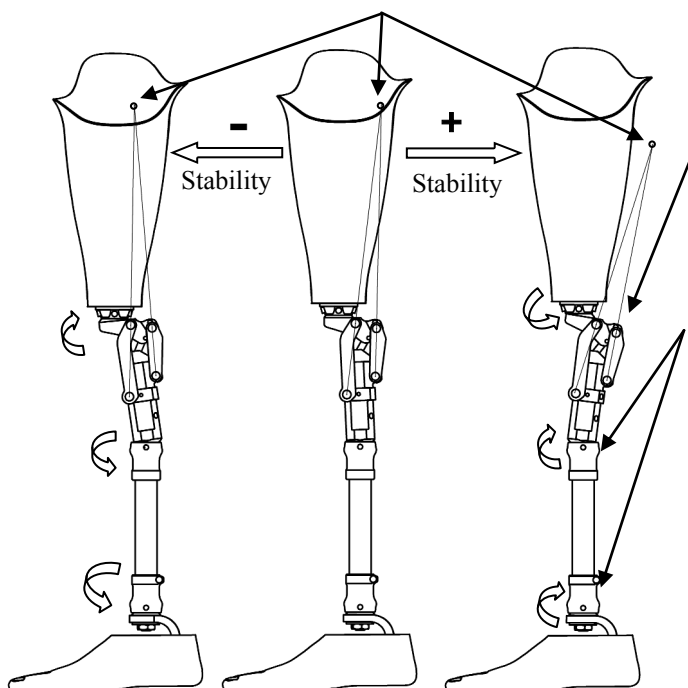Motion of the instantaneous center of rotation of the knee

1- To increase stability, tilt the knee as indicated on the opposite sketch (to move the instantaneous center of rotation backwards) using the upper and lower connectors.
To decrease stability, proceed conversely.

2- Use the intermediate connectors to re-position the foot according to the recommended alignments.

The 1P200-KD model (connection with a M36x1,50 screw) does not allow the stability adjustment to be changed. Therefore the socket must be made with accuracy.

Walking pace adjustment

The 1P200/1P200-KD knee comes adjusted for medium walking pace.

To adapt the initial adjustments to the patient's walking pace, adjust the knobs using a 2.5 mm hex wrench :

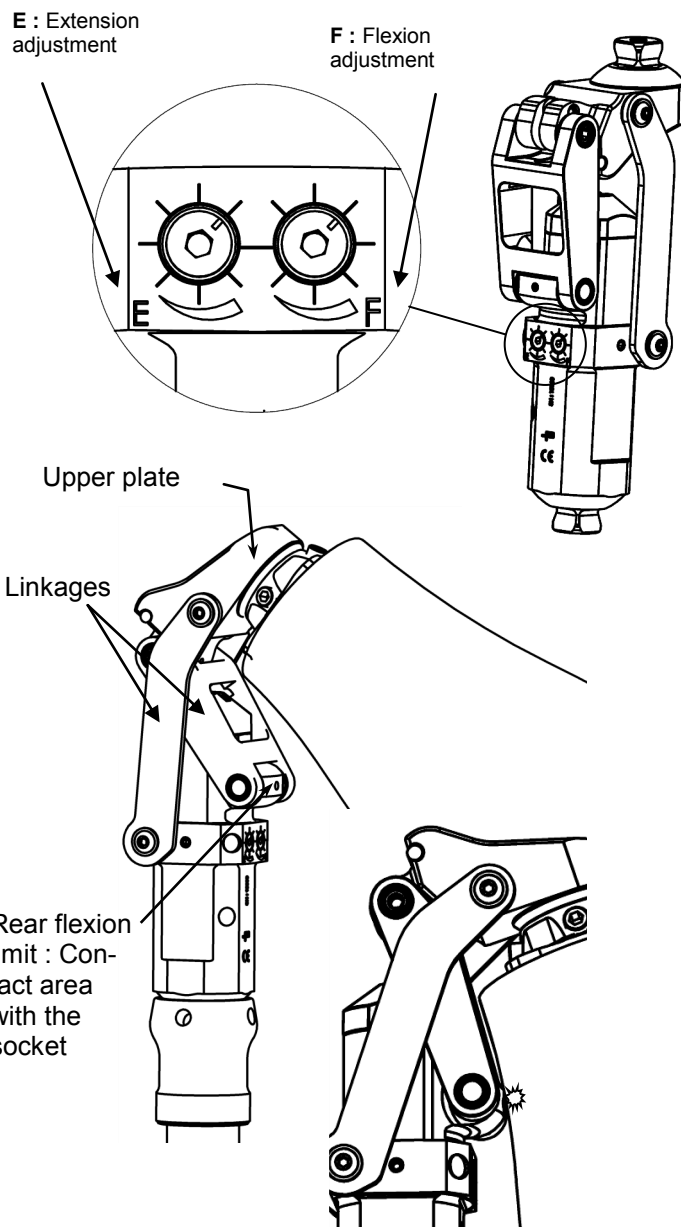
1) To decrease the flexion angle in swing phase, screw on the right knob (F). Proceed conversely to increase the flexion angle.

2) To decrease the impact at the end of extension, screw on the left knob (E).

 When the extension knob adjustment is finalized, check that the knee can reach full extension with low walking speeds (risk of falling).

Flexion limit

When the patient is crouched or kneeled, the socket must get in contact only with the flexion limit of the lower axis of the rear linkage. The socket must touch neither the rear linkage itself nor the plate, because this could strongly damage the knee.

Maintenance:

Check at least once a year the knee functions and make the necessary maintenance operations.

Spare stops : EE014 (two are necessary).

 Use a silicon spray to eliminate possible noises due to cosmetic cover friction. Do not use talcum powder because it damages the mechanical parts, which may lead to function defects and risk of falling for the patient. **The application of talcum powder will render all claims against PROTEOR null and void.**

 Except from knobs (E) and (F), screwing out or loosening any other screws of the knee is forbidden.

 Do not grease the knee roll bearings because this could damage them very soon.

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